Deutsch

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English

Important using information/ please read carefully before start using the instrument!!!

Gaining this instrument you decided for a product of highest quality made in Germany. In order to secure its function and security during a long time please pay attention to following articles.

First using of new instruments

All instruments from **Dufner Instrumente GmbH** will be delivered unsteril and must be cleaned and sterilized from the user before using, exceptionally all instruments, marked with "steril".

Sterilization

All instruments concerning this using information can be autoclaved. Plasma, gamma and steam sterilization is not allowed for instruments with plastic parts, as the plastic will be destroyed.

Safety control

Before every using of the instrument a visible control should be made. Please pay attention to scratches, breakings or mechanical malfunctions. Please see especially mistakes on critical places, as tips, cuttings, ends, ratchet and all movable parts.

MICASEPT-Instruments

The sealing of the MICASEPT instruments must be controlled from DUFNER at latest after two years.

Cleaning and Care of the instruments

After using, the instruments should be cleaned and disinfected immediately. On this way you should pay attention to critical parts as ratchet, cuttings, slits and other places which are difficult to reach (please see ultrasound cleaning).

On this way you should pay attention on critical parts as ratchet, cuttings, slits and other places which are difficult to reach.

Dismantable instruments must be cleaned in principal in dismantled condition.

After cleaning and rinsing of the instruments they must be dried immediately.

All instruments must be treated with suitable and physiological uncritical oil **after** every cleaning and **before** sterilization. Please again, pay attention to the special care of the critical places.

Ultrasound cleaning:

The ultrasound cleaning is to prefer to the mechanical cleaning. Especially for the sensitive endoscopy instruments this procedure is particularly suitable.

- the instruments have to be covered complete by the cleaning liquid during the cleaning procedure
- the cleaning time should be at least 3 minutes with a frequency of 35 KHz
- the instruments have to be rinsed thoroughly manual or mechanical, the best would be rinsing with water complete free of salt

On minimal invasive instruments, especially MICASEPT instruments the movable jaw tips must be cleaned sufficient in ultrasound. During cleaning procedure the jaw part should be closed and opened several times. If this will be done immediately after operation, there could be residues in the front part of the jaws and correct functioning of the sealing could be avoided.

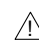
Pay attention to the instructions of the ultrasound unit manufacturer!

Handling

Handle your instruments always with the necessary care. Take steps for protection against damage during transport, cleaning, care, sterilization and storage.

Do not contact your instruments with aggressive materials, as this could lead to rust forming and injury of the functions until complete uselessness of the instruments.

This is especially valid for using of acids or burning detergents.

 **Please pay attention to the instructions of the detergent and disinfectant manufacturers!**

Storage

All instruments should be stored dry – not in metal boxes (exceptionally refined steel or aluminium boxes). Please avoid direct sun irradiation.

Improper handling of the instruments, delivered from Dufner Instrumente GmbH, leads to exclusion of all warranty claims!

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**Important
using direction
for
DUFNER[®] instruments**

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**Dufner - we offer, what you need:
Innovation, reliablitiy, service.**

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